

Introducing *AG INDUSTRIES* **Boomerang Gel Pad**



TO USE:

- Wash the Boomerang Gel Pad™ before each use as directed.
- Place the Boomerang Gel Pad™ on bridge of the nose as shown.
- Fit your mask as usual over the gel pad.
- Replacement recommended every 30 days.

RECOMMENDED CLEANING:

- Wash in warm water using mild soap or with alcohol-based hand sanitizer.
- Rinse with warm water and allow to air-dry.
- Keep out of direct sunlight.

SOLD AS:

Petite/Small

Part # **AGGELPAD-PS**

Medium/Large

Part # **AGGELPAD-ML**

Pack with both sizes

Part # **AGGELPAD-PK**

HCPCS # A7032



Please contact customer service or your sales representative for more information on this product.